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INDICATION

The ML acts as an additive to the particular ceramic modelling fluid to lengthen the processing time of mixed layering ceramics so that the ceramic already applied to the frame does not dry out as quickly.

Its addition also makes the mixed ceramic slightly more plastic so that it can be applied more specifically.

- for extensive restorations
- for complex correction firings
- to make the ceramic slightly easier to model

USE

For all-purpose use with all layering ceramics.

DOSING

Add one drop of Moisturing Liquid per approx. 10 grams of mixed layering ceramic and mix thoroughly.

FIRING GUIDE

The pre-drying time and the closure time each need to be extended by one minute in order to ensure the liquid evaporates.

SAFETY

- If in contact with skin: wash off with plenty of water.
- If in contact with eyes: rinse out with plenty of water and with the eyelids held open.
- If swallowed: drink plenty of water.
Consult a doctor if you feel unwell.
- Protective gear: gloves, protective glasses and protective clothing.

NOTE

The directions for use are constantly updated to reflect the latest developments. We recommend you read the directions for use before using a new pack.

IMPORTANT

Original pack. If pack is opened, no refund can be offered.

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